

Setting the movement parameters




EN




Press the On/Off switch next to the power supply cable.

Press the  button to **start** movement with the settings used previously.

Press the  button to **stop** movement.

Press the  button to **restart** movement in the opposite direction.

To change the **extension limit**: press the  button, then the  or  buttons within 5 seconds to set the limit.


To change the **flexion limit**: press the  button, then the  or  buttons within 5 seconds to set the limit.

Locking/unlocking the handle,

Press the  and  buttons at the same time.

Installing the patient

- Stop the machine in a position that is **comfortable for the patient**.
- Loosen both buttons (1) and extend the **foot plate** (2) as far as it will go.
- Measure the patient's **femur length** (L).
- Loosen both buttons (3) then **transfer the measurement** to the graduated scale (4), tighten both buttons (3).
- **Install the patient on the machine.**
- Bring the board (2) into contact with the patient's foot, then tighten both buttons (1).

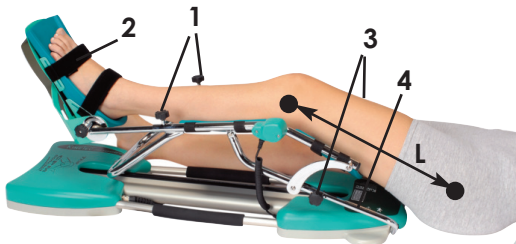

Refer to the user manual for more detail.